



Abbott



CONNECT MORE. SHARE MORE. WORRY LESS.

Duncan Mills Informatics Manager for Europe Middle East APAC,

duncan.mills@abbott.com

+447717366543


Decemeber | 2023



Point of Care QC /EQA Management



AegisPOC

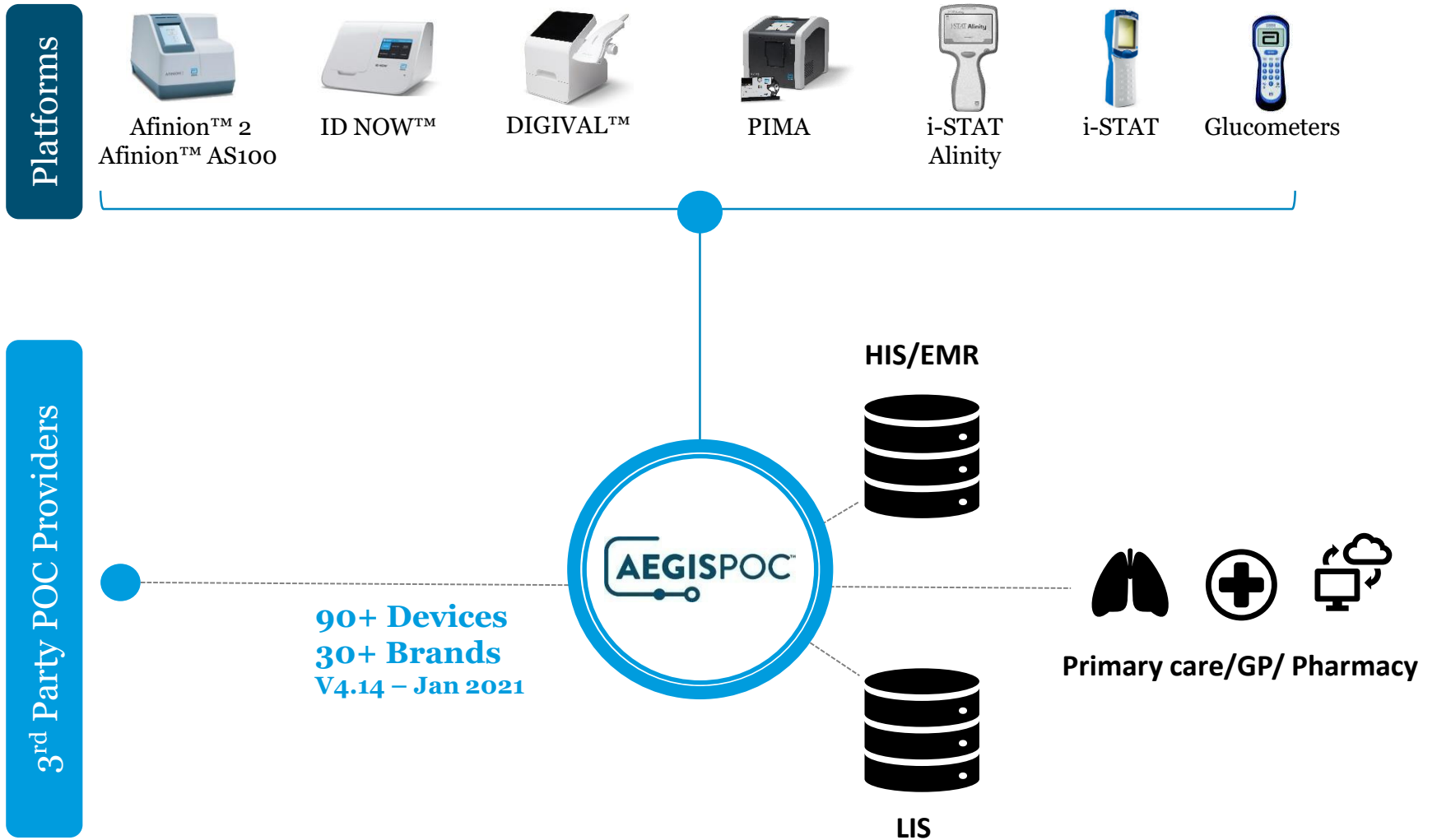
Open and Vendor-Neutral




AegisPOC | Operator Certification

AEGISPOC **Abbott**

789 Border Collie  

AUTO-CERTIFICATION CRITERIA  PRINT

Auto Certification Configuration Name:



Abbott i-STAT - Instrument Certification

Once an Operator Meets All Certification Criteria, Their Certification Should be Recertified for (Number of Months):



First Re-Certification	<input type="text" value="3"/>
Subsequent Re-Certifications	<input type="text" value="12"/>
Date to Re-Certify From	<input type="text" value="Today"/>

Required	Certification Criteria	Option
<input checked="" type="checkbox"/>	Require a Passing Quiz	
	How Long Since Last Quiz	<input type="text" value="Days"/> <input type="text" value="30"/>
<input type="checkbox"/>	Number of Successful QC Samples	<input type="text" value="1"/>
	How Long Since QC Samples	<input type="text" value="Days"/> <input type="text" value="30"/>
	<input type="checkbox"/> Require at Least One Sample of Level	<input type="text" value="1"/>
	<input type="checkbox"/> Require at Least One Sample of Level	<input type="text" value="2"/>
	<input type="checkbox"/> Require at Least One Sample of Level	<input type="text" value="3"/>

OPERATORS

STATUS PATIENTS RESULTS QC STOCK POC MANUAL TEST REPORTS INSTRUMENTS MAINTENANCE LOCATIONS SETUP

AegisPOC | Results

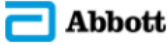

- STATUS
- PATIENTS
- RESULTS**
- QC
- STOCK
- POC
- MANUAL TEST
- REPORTS
- INSTRUMENTS
- MAINTENANCE
- OPERATORS
- LOCATIONS

ALL RESULTS

Facility: Workstation: ID NO: Instrument... State... Patient All ID Now

	New	View-Edit	Delete	Export	Delegate	Report	Review	Reassign	Stats	Save
<input type="checkbox"/>	HIS	Date and Time	Receipt Date	Operator	Account #	Med Rec #	Last Name	First		
1	<input type="checkbox"/>	Not Sent 03/08/2021 14:39	03/08/2021 14:39	123456	Pat1Account	Pat1MRN	Doe 1	John		
2	<input type="checkbox"/>	Not Sent 03/08/2021 14:37	03/08/2021 14:39	test	Pat1Account	Pat1MRN	Doe 1	John		
3	<input type="checkbox"/>	Not Sent 03/08/2021 13:39	03/08/2021 14:39	System	Pat1Account	Pat1MRN	Doe 1	John		
4	<input type="checkbox"/>	Not Sent 03/08/2021 13:37	03/08/2021 14:39	test	Pat1Account	Pat1MRN	Doe 1	John		
5	<input type="checkbox"/>	Not Sent 03/08/2021 12:39	03/08/2021 14:39	LDS	Pat1Account	Pat1MRN	Doe 1	John		
6	<input type="checkbox"/>	Not Sent 03/08/2021 12:37	03/08/2021 14:39	test	Pat1Account	Pat1MRN	Doe 1	John		
7	<input type="checkbox"/>	Not Sent 03/08/2021 11:39	03/08/2021 14:39	LDS	Pat1Account	Pat1MRN	Doe 1	John		
8	<input type="checkbox"/>	Not Sent 03/08/2021 11:37	03/08/2021 14:39	franck	Pat1Account	Pat1MRN	Doe 1	John		
9	<input type="checkbox"/>	Not Sent 03/08/2021 10:39	03/08/2021 14:39	LDS	Pat1Account	Pat1MRN	Doe 1	John		
10	<input type="checkbox"/>	Not Sent 03/08/2021 10:37	03/08/2021 14:39	LDS	Pat1Account	Pat1MRN	Doe 1	John		
11	<input type="checkbox"/>	Not Sent 03/08/2021 09:39	03/08/2021 14:39	123456	Pat1Account	Pat1MRN	Doe 1	John		
12	<input type="checkbox"/>	Not Sent 03/08/2021 09:37	03/08/2021 14:39	LDS	Pat1Account	Pat1MRN	Doe 1	John		
13	<input type="checkbox"/>	Not Sent 03/08/2021 08:39	03/08/2021 14:39	System	Pat1Account	Pat1MRN	Doe 1	John		
14	<input type="checkbox"/>	Not Sent 03/08/2021 08:37	03/08/2021 14:39	123456	Pat1Account	Pat1MRN	Doe 1	John		
15	<input type="checkbox"/>	Not Sent 03/08/2021 07:39	03/08/2021 14:39	franck	Pat1Account	Pat1MRN	Doe 1	John		
16	<input type="checkbox"/>	Not Sent 03/08/2021 07:37	03/08/2021 14:39	franck	Pat1Account	Pat1MRN	Doe 1	John		
17	<input type="checkbox"/>	Not Sent 03/08/2021 06:39	03/08/2021 14:39	System	Pat1Account	Pat1MRN	Doe 1	John		
18	<input type="checkbox"/>	Not Sent 03/08/2021 06:37	03/08/2021 14:39	System	Pat1Account	Pat1MRN	Doe 1	John		
19	<input type="checkbox"/>	Not Sent 03/08/2021 05:39	03/08/2021 14:39	test	Pat1Account	Pat1MRN	Doe 1	John		
20	<input type="checkbox"/>	Not Sent 03/08/2021 05:37	03/08/2021 14:39	123456	Pat1Account	Pat1MRN	Doe 1	John		
21	<input type="checkbox"/>	Not Sent 03/08/2021 04:39	03/08/2021 14:39	test	Pat1Account	Pat1MRN	Doe 1	John		


AegisPOC | Quality Control

AEGISPOC **Abbott**


789 Border Collie

QUALITY CONTROL PRINT


Overview

 Queued QC Results (This Workstation): 0
Queued QC Results (All Workstations): 0
Total last 24 hours: 0
Total QC results: 108.264


Intelligent Quality Management

 [Delta Charts](#)
[Corrective Action Report](#)
[CVP Results](#)


View QC Results and Graphs

 [Levey-Jennings Graphs](#)
[QC-Data List-Interactive \(All QC Results\)](#)
[Proficiency Tests](#)
[Linearity Graphs](#)


Reagents

 [Add New Material](#)
[List of Materials](#)
[Instrument Material Relationship](#)

Manual Entry



AegisPOC | Quality Control Chart

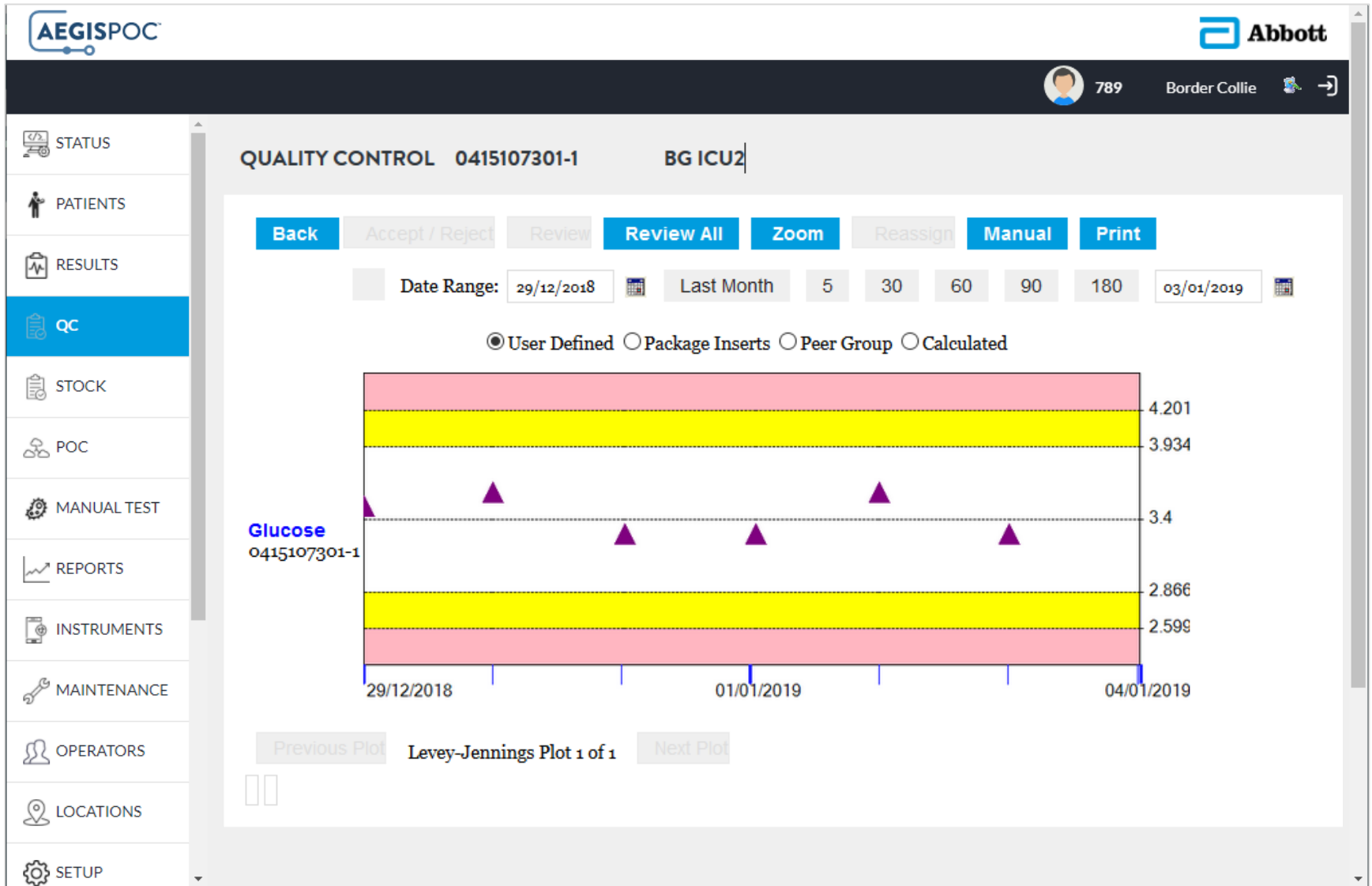



AegisPOC | POC

The screenshot shows the AegisPOC POC dashboard. At the top left is the AEGISPOC logo. At the top right is the Abbott logo. Below the logos is a dark navigation bar containing a user profile icon with the number 789, the name Border Collie, and a search icon. A left sidebar contains a vertical menu with icons and labels for STATUS, PATIENTS, RESULTS, QC, STOCK, POC (highlighted in blue), MANUAL TEST, REPORTS, INSTRUMENTS, MAINTENANCE, OPERATORS, LOCATIONS, and SETUP. The main content area is titled POC and features a PRINT button in the top right corner. The main area is divided into six sections, each with an icon and a list of items:

- Patient Manual Test Entry** (Syringe icon): FOB, FOCB, Mono, StrepA
- Quality Control Manual Test Entry** (Line graph icon): FOBQ, FOCB, Mono, StrepAQ
- Proficiency Manual Test Entry** (Star medal icon): FOBR, Mono, StrepAR
- Quizzes and Certification** (Certificate icon): [Take Quiz](#), [My Certifications](#)
- Learning Materials** (Stack of books icon): [Device list](#), [iSTAT brochure](#)

AegisPOC | Manual Test Entry



- STATUS
- PATIENTS
- RESULTS
- QC
- STOCK
- POC
- MANUAL TEST**
- REPORTS
- INSTRUMENTS
- MAINTENANCE
- OPERATORS
- LOCATIONS
- SETUP
- LOG

Visit#: 991093365	MRN#: 771235	Facility: Brazil
Name: PatLast1235 PatFirst1235	Station: C.H Sao Paulo	
DOB: 20/03/1956	Sex: Male	Instrument: Please Select Instrumen
Physician: PhysFull84	Date/Time: 30/03/2021 14:22:54	Lot:
Location:		

EDIT PATIENT


Result Information

Order Number:

Analyzed by: LDS	SARS-CoV2 Result: Negative
SARS-CoV-2 Ab Result: Negative	SARS-CoV-2 Ag Result: Negative
SARS-CoV-2 Viral Load Result:	SARS-CoV-2 Ab IgM Result: Negative
SARS-CoV-2 AbIgG Result: Negative	INT QC Present: No

Comment:

FINISH **CANCEL**



Choose File

Enable Camera

AegisPOC | Reports

AEGISPOC **Abbott**

789 Border Collie

REPORTS PRINT

- Results**
 - All Queued Results
 - Stock Usage Report
 - Panel Usage
- Quality Control**
 - QC Data List
 - QC Operator Performance
 - Levey-Jennings Plot
 - QC Statistics
 - QC Statistics With Reagents
 - Calibrations
 - Proficiency
 - Linearity
 - Electronic QC (EQC)
 - QC Test Report
- Operators**
 - Certification
 - Quiz Certificate
 - Quiz Review
 - Edit Log
 - Operator List/Setup
 - Operator Competency
 - Device Competency
 - Operator Message
 - Operator Competency Failure Report
- Patients**
 - All Patient Results
 - Critical Patient Results
 - Daily Sample Log
 - Patient History
 - Most Recent Result
 - Oximetry Shunt
 - RxDx
 - Orders List
 - Patient Admission List
- Workstations**
 - Queued Results by Workstation
 - Configurations By Workstation
 - Location Usage Report
 - Data Log by Workstation
 - Test Type Count By Workstation
- Instruments**
 - Information by Instrument Type
 - Instrument Competency
- System**
 - Audit Log
 - Patient Audit Log
 - Maintenance Log
 - Physician



Abbott